

# One Step Drug Screen Test Card

## Package Insert for Single and Multi Drug Screen Test Cards

**This Instruction Sheet is for testing of any combination of Amphetamine, Barbiturates, Benzodiazepines, Cocaine, Methamphetamine, Morphine, Methadone, Phencyclidine and Marijuana.**

*A rapid, one step screening test for the simultaneous, qualitative detection of multiple drugs and drug metabolites in human urine.*

*For healthcare professionals including professionals at point of care sites.*

*For in vitro diagnostic use only.*

### INTENDED USE

The One Step Drug Screen Test Card is a lateral flow chromatographic immunoassay for the qualitative detection of multiple drugs and drug metabolites in urine at the following cut-off concentrations:

Test	Calibrator	Cut-off
<b>Amphetamine (AMP)</b>	<b>D-Amphetamine</b>	<b>1,000 ng/mL</b>
<b>Barbiturates (BAR)</b>	<b>Secobarbital</b>	<b>300 ng/mL</b>
<b>Benzodiazepines (BZO)</b>	<b>Oxazepam</b>	<b>300 ng/mL</b>
<b>Cocaine (COC)</b>	<b>Benzoylcegonine</b>	<b>300 ng/mL</b>
<b>Methamphetamine (mAMP)</b>	<b>D-Methamphetamine</b>	<b>1,000 ng/mL</b>
<b>Morphine (MOP 300 or OPI 300)</b>	<b>Morphine</b>	<b>300 ng/mL</b>
<b>Methadone (MTD)</b>	<b>Methadone</b>	<b>300 ng/mL</b>
<b>Opiates (OPI 2000)</b>	<b>Morphine</b>	<b>2,000 ng/mL</b>
<b>Phencyclidine (PCP)</b>	<b>Phencyclidine</b>	<b>25 ng/mL</b>
<b>Marijuana (THC)</b>	<b>11-nor-<sup>9</sup>-THC-9 COOH</b>	<b>50 ng/mL</b>

The configurations of the One Step Multi-Drug Screen Test Card come with any combination of the above listed drug analytes. This assay provides only a preliminary analytical test result. A more specific alternate chemical method must be used in order to obtain a confirmed analytical result. Gas chromatography/mass spectrometry (GC/MS) is the preferred confirmatory method. Clinical consideration and professional judgment should be applied to any drug of abuse test result, particularly when preliminary positive results are used.

### SUMMARY

#### AMPHETAMINE (AMP)

Amphetamine is a Schedule II controlled substance available by prescription (Dexedrine®) and is also available on the illicit market. Amphetamines are a class of potent sympathomimetic agents with therapeutic applications. They are chemically related to the human body's natural catecholamines: epinephrine and norepinephrine. Acute higher doses lead to enhanced stimulation of the central nervous system and induce euphoria, alertness, reduced appetite, and a sense of increased energy and power. Cardiovascular responses to amphetamines include increased blood pressure and cardiac arrhythmias. More acute responses produce anxiety, paranoia, hallucinations, and psychotic behavior. The effects of amphetamines generally last 2-4 hours following use, and the drug has a half-life of 4-24 hours in the body. About 30% of amphetamines are excreted in the urine in unchanged form, with the remainder as hydroxylated and deaminated derivatives.

The AMP One Step Amphetamine Test Strip is a rapid urine screening test that can be performed without the use of an instrument. The test utilizes a monoclonal antibody to selectively detect elevated levels of amphetamine in urine. The AMP One Step Amphetamine Test Strip yields a positive result when amphetamines in urine exceed 1,000 ng/mL.

#### BARBITURATES (BAR)

Barbiturates are central nervous system depressants. They are used therapeutically as sedatives, hypnotics, and anticonvulsants. Barbiturates are almost always taken orally as capsules or tablets. The effects resemble those of intoxication with alcohol. Chronic use of barbiturates leads to tolerance and physical dependence.

Short acting barbiturates taken at 400 mg/day for 2-3 months produces a clinically significant degree of physical dependence. Withdrawal symptoms experienced during periods of drug abstinence can be severe enough to cause death.

Only a small amount (less than 5%) of most barbiturates are excreted unaltered in the urine.

The approximate detection time limits for barbiturates are:

Short acting (e.g. Secobarbital)	100 mg PO (oral)	4.5 days
Long acting (e.g. Phenobarbital)	400 mg PO (oral)	7 days1

The BAR One Step Barbiturates Test Strip is a rapid urine-screening test that can be performed without the use of an instrument. The test utilizes a monoclonal antibody to selectively detect elevated levels of barbiturates in urine. The BAR One Step Barbiturates Test Strip yields a positive result when the barbiturates in urine exceed the cut-off level.

#### BENZODIAZEPINES (BZO)

Benzodiazepines are medications that are frequently prescribed for the symptomatic treatment of anxiety and sleep disorders. They produce their effects via specific receptors involving a neurochemical called gamma aminobutyric acid (GABA). Because they are safer and more effective, benzodiazepines have replaced barbiturates in the treatment of both anxiety and insomnia. Benzodiazepines are also used as sedatives before some surgical and medical procedures, and for the treatment of seizure disorders and alcohol withdrawal.

Risk of physical dependence increases if benzodiazepines are taken regularly (e.g., daily) for more than a few months, especially at higher than normal doses. Stopping abruptly can bring on such symptoms as trouble sleeping, gastrointestinal upset, feeling unwell, loss of appetite, sweating, trembling, weakness, anxiety and changes in perception.

Only trace amounts (less than 1%) of most benzodiazepines are excreted unaltered in the urine; most of the concentration in urine is conjugated drug. The detection period for the benzodiazepines in the urine is 3-7 days.

The BZO One Step Benzodiazepines Test Strip is a rapid urine-screening test that can be performed without the use of an instrument. The test utilizes the antibody to selectively detect elevated levels of benzodiazepines in urine. The BZO One Step Benzodiazepines Test Strip yields a positive result when the benzodiazepines in urine exceeds cut-off concentration.

#### COCAINE (COC)

Cocaine is a potent central nervous system (CNS) stimulant and a local anesthetic. Initially, it brings about extreme energy and restlessness while gradually resulting in tremors, oversensitivity and spasms. In large amounts, cocaine causes fever, unresponsiveness, and difficulty in breathing and unconsciousness.

Cocaine is often self-administered by nasal inhalation, intravenous injection and free-base smoking. It is excreted in the urine in a short time primarily as benzoylcegonine<sup>1,2</sup>. Benzoylcegonine, a major metabolite of cocaine, has a longer biological half-life (5-8 hours) than cocaine (0.5-1.5 hours), and can generally be detected for 24-48 hours after cocaine exposure<sup>2</sup>. The COC One Step Cocaine Test Strip is a rapid urine screening test that can be performed without the use of an instrument. The test utilizes a monoclonal antibody to selectively detect elevated levels of cocaine metabolite in urine. The COC One Step Cocaine Test Strip yields a positive result when the cocaine metabolite in urine exceeds 300 ng/mL. This is the suggested screening cut-off for positive specimens set by the Substance Abuse and Mental Health Services Administration (SAMHSA, USA).

#### MARIJUANA (THC)

THC (Δ<sup>9</sup>-tetrahydrocannabinol) is the primary active ingredient in cannabinoids (marijuana). When smoked or orally administered, it produces euphoric effects. Users have impaired short term memory and slowed learning. They may also experience transient episodes of confusion and anxiety. Long term relatively heavy use may be associated with behavioral disorders. The peak effect of smoking marijuana occurs in 20-30 minutes and the duration is 90-120 minutes after one cigarette. Elevated levels of urinary metabolites are found within hours of exposure and remain detectable for 3-10 days after smoking. The main metabolite excreted in the urine is 11-nor-Δ<sup>9</sup>-tetrahydrocannabinol-9-carboxylic acid (Δ<sup>9</sup>-THC-COOH).

The THC One Step Marijuana Test Strip is a rapid urine screening test that can be performed without the use of an instrument. The test utilizes a monoclonal antibody to selectively detect elevated levels of marijuana in urine. The THC One Step Marijuana Test Strip yields a positive result when the concentration of marijuana in urine exceeds 50 ng/mL. This is the suggested screening cut-off for positive specimens set by the Substance Abuse and Mental Health Services Administration (SAMHSA, USA).<sup>3</sup>

#### METHADONE (MTD)

Methadone is a narcotic pain reliever for medium to severe pain. It is also used in the treatment of heroin (opiate dependence: Vicodin, Percocet, Morphine, etc.) addiction. Oral methadone is very different than IV methadone. Oral methadone is partially stored in the liver for later use. IV methadone acts more like heroin. In most states you must go to a pain clinic

or a methadone maintenance clinic to be prescribed methadone.

Methadone is a long acting pain reliever producing effects that last from twelve to forty-eight hours. Ideally, methadone frees the client from the pressures of obtaining illegal heroin, from the dangers of injection, and from the emotional roller coaster that most opiates produce. Methadone, if taken for long periods and at large doses, can lead to a very long withdrawal period. The withdrawals from methadone are more prolonged and troublesome than those provoked by heroin cessation, yet the substitution and phased removal of methadone is an acceptable method of detoxification for patients and therapists.<sup>1</sup>

The MTD One Step Methadone Test Strip is a rapid urine screening test that can be performed without the use of an instrument. The test utilizes a monoclonal antibody to selectively detect elevated levels of methadone in urine. The MTD One Step Methadone Test Strip yields a positive result when the methadone in urine exceeds 300 ng/mL.

#### METHAMPHETAMINE (mAMP)

Methamphetamine is an addictive stimulant drug that strongly activates certain systems in the brain. Methamphetamine is closely related chemically to amphetamine, but the central nervous system effects of methamphetamine are greater. Methamphetamine is made in illegal laboratories and has a high potential for abuse and dependence. The drug can be taken orally injected, or inhaled. Acute higher doses lead to enhanced stimulation of the central nervous system and induce euphoria, alertness, reduced appetite, and a sense of increased energy and power. Cardiovascular responses to methamphetamine include increased blood pressure and cardiac arrhythmias. More acute responses produce anxiety, paranoia, hallucinations, psychotic behavior, and eventually, depression and exhaustion.

The effects of methamphetamine generally last 24 hours and the drug has a half-life of 9-24 hours in the body. Methamphetamine is excreted in the urine primarily as amphetamine and oxidized and deaminated derivatives. However, 10-20% of methamphetamine is excreted unchanged. Thus, the presence of the parent compound in the urine indicates methamphetamine use. Methamphetamine is generally detectable in the urine for 35 days, depending on urine pH level.

The mAMP One Step Methamphetamine Test Strip is a rapid urine screening test that can be performed without the use of an instrument. The test utilizes a monoclonal antibody to selectively detect elevated levels of methamphetamine in urine. The mAMP One Step Methamphetamine Test Strip yields a positive result when the methamphetamine in urine exceeds 1,000 ng/mL.

#### OPIATE (MOP 300 or OPI 300)

Opiate refers to any drug that is derived from the opium poppy, including the natural products morphine and codeine, and the semi-synthetic drugs such as heroin. Opioid is more general referring to any drug that acts on the opioid receptor.

Opioid analgesics comprise a large group of substances which control pain by depressing the central nervous system. Large doses of morphine can produce higher tolerance levels, physiological dependency in users, and may lead to substance abuse. Morphine is excreted unmetabolized, and is also the major metabolic product of codeine and heroin. Morphine is detectable in the urine for several days after an opiate dose (1).

The MOP One Step Opiate Test Strip is a rapid urine screening test that can be performed without the use of an instrument. The test utilizes a monoclonal antibody to selectively detect elevated levels of opiates in urine. The MOP One Step Opiate Test Strip yields a positive result when the concentration of opiate exceeds the 300 ng/mL cut-off level.

#### OPIATE (2000)

Opiate refers to any drug that is derived from the opium poppy, including the natural products morphine and codeine, and the semi-synthetic drugs such as heroin. Opioid is more general referring to any drug that acts on the opioid receptor.

Opioid analgesics comprise a large group of substances which control pain by depressing the central nervous system. Large doses of morphine can produce higher tolerance levels, physiological dependency in users, and may lead to substance abuse. Morphine is excreted unmetabolized, and is also the major metabolic product of codeine and heroin. Morphine is detectable in the urine for several days after an opiate dose.<sup>4</sup>

The OPI One Step Opiate Test Strip is a rapid urine screening test that can be performed without the use of an instrument. The test utilizes a monoclonal antibody to selectively detect elevated levels of morphine in urine. The OPI One Step Opiate Test Strip yields a positive result when the morphine in urine exceeds 2,000 ng/mL. This is the suggested screening cut-off for positive specimens set by the Substance Abuse and Mental Health Services Administration (SAMHSA USA).

#### PHENCYCLIDINE (PCP)

Phencyclidine, also known as PCP or Angel Dust, is a hallucinogen that was first marketed as a

urgical anesthetic in the 1950 s. It was removed from the market because patients receiving it became delirious and experienced hallucinations.

Phencyclidine is used in powder, capsule, and tablet form. The powder is either snorted or smoked after mixing it with marijuana or vegetable matter. Phencyclidine is most commonly administered by inhalation but can be used intravenously, intra-nasally, and orally. After low doses, the user thinks and acts swiftly and experiences mood swings from euphoria to depression. Self-injurious behavior is one of the devastating effects of Phencyclidine.

PCP can be found in urine within 4 to 6 hours after use and will remain in urine for 7 to 14 days, depending on factors such as metabolic rate, user's age, weight, activity, and diet.<sup>5</sup> Phencyclidine is excreted in the urine as an unchanged drug (4% to 19%) and conjugated metabolites (25% to 30%).<sup>6</sup>

The PCP One Step Phencyclidine Test Strip is a rapid urine screening test that can be performed without the use of an instrument. The test utilizes a monoclonal antibody to selectively detect elevated levels of phencyclidine metabolite in urine. The PCP One Step Phencyclidine Test Strip yields a positive result when the phencyclidine metabolite in urine exceeds 25 ng/mL. This is the suggested screening cut-off for positive specimens set by the Substance Abuse and Mental Health Services Administration (SAMHSA, USA).

## PRINCIPLE

The One Step Drug Screen Test Card is an immunoassay based on the principle of competitive binding. Drugs which may be present in the urine specimen compete against their respective drug conjugate for binding sites on their specific antibody.

During testing, a urine specimen migrates upward by capillary action. A drug, if present in the urine specimen below its cut-off concentration, will not saturate the binding sites of its specific antibody. The antibody will then react with the drug-protein conjugate and a visible colored line will show up in the test line region of the specific drug strip. The presence of drug above the cut-off concentration will saturate all the binding sites of the antibody. Therefore, the colored line will not form in the test line region.

A drug-positive urine specimen will not generate a colored line in the specific test line region of the strip because of drug competition, while a drug-negative urine specimen will generate a line in the test line region because of the absence of drug competition.

To serve as a procedural control, a colored line will always appear at the control line region, indicating that proper volume of specimen has been added and membrane wicking has occurred.

## REAGENTS

The test contains a membrane strip coated with drug-protein conjugates (purified bovine albumin) on the test line, a goat polyclonal antibody against gold-protein conjugate at the control line, and a dye pad which contains colloidal gold particles coated with mouse monoclonal antibody specific to Amphetamine, Cocaine, Methamphetamine, Morphine, THC, Phencyclidine, Benzodiazepine, Methadone or Barbiturate.

## PRECAUTIONS

For healthcare professionals including professionals at point of care sites.

For *in vitro* diagnostic use only. Do not use after the expiration date.

The test panel should remain in the sealed pouch until use.

All specimens should be considered potentially hazardous and handled in the same manner as an infectious agent.

The used test card should be discarded according to federal, state and local regulations.

## STORAGE AND STABILITY

Store as packaged in the sealed pouch at 2-30°C. The test strip is stable through the expiration date printed on the sealed pouch. The test strips must remain in the sealed pouch until use. DO NOT FREEZE. Do not use beyond the expiration date.

## SPECIMEN COLLECTION AND PREPARATION

### Urine Assay

The urine specimen must be collected in a clean and dry container. Urine collected at any time of the day may be used. Urine specimens exhibiting visible precipitates should be centrifuged, filtered, or allowed to settle to obtain a clear specimen for testing.

### Specimen Storage

Urine specimens may be stored at 28°C for up to 48 hours prior to testing. For prolonged storage, specimens may be frozen and stored below -20°C. Frozen specimens should be thawed and mixed well before testing.

## MATERIALS

### Materials Provided

- Test cards
- Package insert


### Materials Required But Not Provided

- Specimen collection container
- Timer
- External controls

## DIRECTIONS FOR USE

Allow the test panel, urine specimen, and/or controls to equilibrate to room temperature (15-30° C) prior to testing.

1. Bring the pouch to room temperature before opening it. Remove the test panel from the sealed pouch and use it as soon as possible.
2. Remove the cap from the end of the test card. With arrows pointing toward the urine specimen, immerse the strip(s) of the test card vertically in the urine specimen for at least 10-15 seconds. Do not pass the arrow(s) on the test panel when immersing the panel. See the illustration below.
3. Place the test card on a non-absorbent flat surface, start the timer and wait for the red line(s) to appear. The results should be read at 5 minutes. Do not interpret results after 10 minutes.

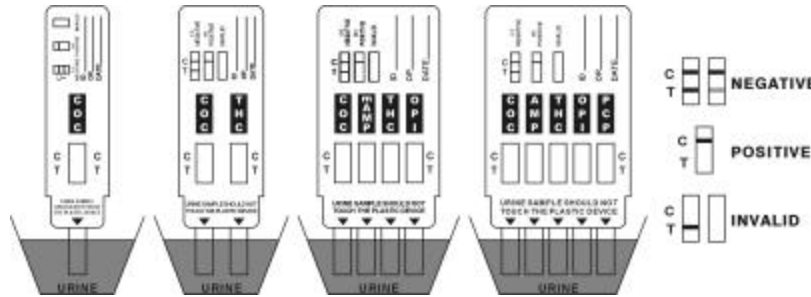

## INTERPRETATION OF RESULTS

(Please refer to the illustration above)

**NEGATIVE:** \* **Two lines appear.** One red line should be in the control region (C), and another apparent red or pink line adjacent should be in the test region (T). This negative result indicates that the drug concentration is below the detectable level.

**\*NOTE:** The shade of red in the test line region (T) will vary, but it should be considered negative whenever there is even a faint pink line.

**POSITIVE:** **One red line appears in the control region (C). No line appears in the test region (T).** This positive result indicates that the drug concentration is above the detectable level.

**INVALID: Control line fails to appear.** Insufficient specimen volume or incorrect procedural techniques are the most likely reasons for control line failure. Review the procedure and repeat the test using a new test panel. If the problem persists, discontinue using the lot immediately and contact your local distributor.

## QUALITY CONTROL

A procedural control is included in the test. A red line appearing in the control region (C) is considered an internal procedural control. It confirms sufficient specimen volume, adequate membrane wicking and correct procedural technique.

Control standards are not supplied with this kit. However, it is recommended that positive and negative controls be tested as good laboratory practice to confirm the test procedure and to verify proper test performance.

## LIMITATIONS

1. The One Step Drug Screen Test Card provides only a qualitative, preliminary analytical result. A secondary analytical method must be used to obtain a confirmed result. Gas chromatography and mass spectrometry (GC/MS) is the preferred confirmatory method.<sup>3,4,7</sup>
2. There is a possibility that technical or procedural errors, as well as other interfering substances in the urine specimen may cause erroneous results.
3. Adulterants, such as bleach and/or alum, in urine specimens may produce erroneous results regardless of the analytical method used. If adulteration is suspected, the test should be

repeated with another urine specimen.

4. A Positive result does not indicate level or intoxication, administration route or concentration in urine.
5. A Negative result may not necessarily indicate drug-free urine. Negative results can be obtained when drug is present but below the cut-off level of the test.
6. Test does not distinguish between drugs of abuse and certain medications.

## PERFORMANCE CHARACTERISTICS

### Accuracy

A side-by-side comparison was conducted using the One Step Single Drug Test Strip and commercially available drug rapid tests. Testing was performed on approximately 1,700 specimens previously collected from subjects presenting for Drug Screen Testing. Presumptive positive results were confirmed by GC/MS. The following compounds were quantified by GC/MS and contributed to the total amount of drugs found in presumptive positive urine samples tested.

Test	Compounds Contributed to the Totals of GC/MS
AMP	Amphetamine
BAR	Secobarbital, Butalbital, Phenobarbital, Pentobarbital
BZO	Oxazepam, Nordiazepam, a-OH-Alprazolam, Desakyl-frazeepam
COC	Benzoylcegonine
mAMP	Methamphetamine
MTD	Methadone
OPI	Morphine, Codeine
PCP	Phencyclidine
THC	11-nor-9-carboxy-delta-9-tetrahydrocannabinol

The following results are tabulated from these clinical studies:

Method	GC/MS						
	Neg.	Neg. (< -25% cutoff)	Near cutoff neg. (-25% cutoff to cutoff)	Near cutoff pos. (cutoff to +25% cutoff)	Pos. (> +25% cutoff)	% agreement with GC/MS	
AMP	Positive	0	1	8	18	114	97%
	Negative	149	1	5	4	0	95%
BAR	Positive	0	0	4	5	117	92%
	Negative	150	1	5	1	9	98%
BZO	Positive	0	7	1	5	26	97%
	Negative	149	7	1	3	1	95%
COC	Positive	0	2	15	16	103	98%
	Negative	150	5	7	1	1	91%
mAMP	Positive	0	0	10	9	126	99%
	Negative	150	0	4	1	0	94%

MTD	Positive	0	0	10	10	112	99%
	Negative	150	17	0	0	1	94%
MOP	Positive	0	2	7	10	131	>99%
	Negative	150	0	0	0	0	94%
OPI	Positive	0	0	16	18	116	>99%
	Negative	150	0	0	0	0	90%
PCP	Positive	0	0	6	10	40	>99%
	Negative	150	6	0	0	0	96%
THC	Positive	0	13	9	12	109	88%
	Negative	150	6	0	0	1	99%

orty (40) clinical samples for each drug were run using each of The One Step Single Drug Test strip by an untrained operator at a Professional Point of Care site. Based on GC/MS data, the perator obtained statistically similar Positive Agreement, Negative Agreement and Overall greement rates as trained laboratory personnel.

#### Precision and Analytical Sensitivity

study was conducted at three physician offices by untrained operators using three different ts of product to demonstrate the within run, between run and between operator precision. An identical panel of coded specimens, containing drugs at the concentration of  $\pm 50\%$  and  $\pm 25\%$  ut-off level, was labeled, blinded and tested at each site. The results are given below:

#### COCAINE (COC)

Benzoyllecgonine conc. (ng/mL)	n per site	Site A		Site B		Site C	
		-	+	-	+	-	+
0	15	14*	0	15	0	15	0
150	15	14	1	15	0	14	1
225	15	4	11	5	10	8	7
375	15	0	15	0	15	0	15
450	15	0	15	0	15	1	14

Note: One invalid result was obtained.

#### AMPHETAMINE (AMP)

Amphetamine conc. (ng/mL)	n per site	Site A		Site B		Site C	
		-	+	-	+	-	+
0	15	15	0	15	0	15	0
500	15	15	0	15	0	14	1
750	15	13	2	11	4	11	4
1,250	15	6	9	4	11	4	11
1,500	15	2	13	1	14	1	14

#### METHAMPHETAMINE (mAMP)

Methamphetamine conc. (ng/mL)	n per site	Site A		Site B		Site C	
		-	+	-	+	-	+
0	15	15	0	15	0	15	0
500	15	15	0	14	1	13	2
750	15	11	4	10	5	10	5
1,250	15	8	7	4	11	6	9
1,500	15	1	14	1	14	0	15

#### MARIJUANA (THC)

11-nor-D9-THC-9-COOH conc. (ng/mL)	n per site	Site A		Site B		Site C	
		-	+	-	+	-	+
0	15	15	0	15	0	15	0
25	15	15	0	15	0	14	1
375	15	9	6	14	1	9	6
62.5	15	2	13	0	15	0	15
75	15	0	15	0	15	0	15

#### OPIATES (OPI 2000)

Morphine conc. (ng/mL)	n per site	Site A		Site B		Site C	
		-	+	-	+	-	+
0	15	15	0	15	0	15	0
1,000	15	15	0	15	0	14	1
1,500	15	13	2	11	4	7	8
2,500	15	4	11	1	14	2	13
3,000	15	0	15	0	15	2	13

#### PHENCYCLIDINE (PCP)

Phencyclidine conc. (ng/mL)	n per site	Site A		Site B		Site C	
		-	+	-	+	-	+
0	15	15	0	15	0	15	0
12.5	15	15	0	14	1	14	1
18.75	15	11	4	13	2	10	5
31.25	15	8	7	5	10	1	14
37.5	15	4	11	0	15	0	15

#### BARBITURATES (BAR)

Secobarbital conc. (ng/mL)	n per site	Site A		Site B		Site C	
		-	+	-	+	-	+
0	15	15	0	15	0	15	0
150	15	13	2	15	0	15	0
225	15	5	10	7	8	10	5
375	15	2	13	5	10	5	10
450	15	0	15	1	14	1	14

#### BENZODIAZEPINES (BZO)

Oxazepam conc. (ng/mL)	n per site	Site A		Site B		Site C	
		-	+	-	+	-	+
0	15	15	0	15	0	15	0
150	15	14	1	14	1	15	0
225	15	11	4	14	1	14	1
375	15	0	15	1	14	3	12
450	15	0	15	0	15	0	15

#### METHADONE (MTD)

Methadone conc. (ng/mL)	n per site	Site A		Site B		Site C	
		-	+	-	+	-	+
0	15	15	0	15	0	15	0
150	15	12	3	15	0	15	0
225	15	8	7	14	1	15	0
375	15	0	15	0	15	1	14
450	15	1	14	0	15	0	15

#### OPIATE 300 (MOP 300 OR OPI 300)

Morphine	n	Site A	Site B	Site C
----------	---	--------	--------	--------

conc. (ng/mL)	per site	-	+	-	+	-	+
0	15	15	0	15	0	15	0
150	15	13	2	13	2	15	0
225	15	3	12	7	8	10	5
375	15	0	15	1	14	0	15
450	15	0	15	0	15	0	15

#### Analytical Specificity

The following table lists the concentration of compounds (ng/mL) that are detected positive in urine by One Step Drug Screen Test Card at 5 minutes.

AMPHETAMINE	ng/mL
D-Amphetamine	1,000
D,L-Amphetamine sulfate	3,000
L-Amphetamine	50,000
(±)3,4-Methylenedioxyamphetamine	2,000
Phentermine	3,000
BARBITURATES	
Secobarbital	300
Amobarbital	300
Alphenol	150
Aprobarbital	200
Butobarbital	75
Butalbital	2,500
Butethal	100
Cyclopentobarbital	600
Pentobarbital	300
Phenobarbital	100
BENZODIAZEPINES	
Oxazepam	300
Alprazolam	196
a-Hydroxyalprazolam	1,262
Bromazepam	1,562
Chlordiazepoxide	1,562
Chlordiazepoxide HCl	781
Clobazam	98
Clonazepam	781
Clorazepate dipotassium	195
Delorazepam	1,562
Desalkylflurazepam	390
Diazepam	195
Estazolam	2,500
Flunitrazepam	390
(±) Lorazepam	1,562
RS-Lorazepam glucuronide	156
Midazolam	12,500
Nitrazepam	98
Norchlordiazepoxide	195
Nordiazepam	390
Oxazepam	300
Temazepam	98
Triazolam	2,500
COCAINE	
Benzoyllecgonine	300
Cocaine HCl	780
Cocaethylene	12,500
Ecgonine HCl	32,000

<b>MARIJUANA (THC)</b>	
11-nor- $\Delta^9$ -THC-9 COOH	50
Cannabinol	20,000
11-nor- $\Delta^8$ -THC-9 COOH	30
$\Delta^8$ -THC	15,000
$\Delta^9$ -THC	15,000
<b>METHADONE</b>	
Methadone	300
Doxylamine	50,000
<b>METHAMPHETAMINE</b>	
D-Methamphetamine	1,000
p-Hydroxymethamphetamine	30,000
L-Methamphetamine	8,000
(±)-3,4-Methylenedioxymethamphetamine	2,000
Mephentermine	50,000
<b>OPIATE 300 (MOP)</b>	
Morphine	300
Codeine	300
Ethylmorphine	6,250
Hydrocodone	50,000
Hydromorphone	3,125
Levophanol	1500
6-Monoacetylmorphine	400
Morphine 3- $\beta$ -D-glucuronide	1,000
Norcodeine	6,250
Normorphone	100,000
Oxycodone	30,000
Oxymorphone	100,000
Procaine	15,000
Thebaine	6,250
<b>OPIATES (2000)</b>	
Morphine	2,000
Codeine	2,000
Ethylmorphine	5,000
Hydrocodone	12,500
Hydromorphone	5,000
Levophanol	75,000
6-Monoacetylmorphine	5,000
Morphine 3- $\beta$ -D-glucuronide	2,000
Norcodeine	12,500
Normorphone	50,000
Oxycodone	25,000
Oxymorphone	25,000
Procaine	150,000
Thebaine	100,000
<b>PCP</b>	
Phencyclidine	25
4-Hydroxyphencyclidine	12,500

#### **Effect of Urinary Specific Gravity**

fteen (15) urine samples of normal, high, and low specific gravity ranges (1.000-1.037) were piked with drugs at 50% below and 50% above cut-off levels respectively. The One Step Drug creen Test Card was tested in duplicate using fifteen drug-free urine and spiked urine samples. he results demonstrate that varying ranges of urinary specific gravity does not affect the test asults.

#### **Effect of the Urinary pH**

he pH of an aliquoted negative urine pool was adjusted to a pH range of 5 to 9 in 1 pH unit crements and spiked with drugs at 50% below and 50% above cut-off levels. The spiked, pH-djusted urine was tested with One Step Drug Screen Test Card. The results demonstrate that

varying ranges of pH does not interfere with the performance of the test.

#### **Cross-Reactivity**

A study was conducted to determine the cross-reactivity of the test with compounds in either drug-free urine or drug positive urine containing Cocaine, Barbiturates, Benzodiazepine, Amphetamine, Methamphetamine, Marijuana, Methadone, Opiate or Phencyclidine. The following compounds show no cross-reactivity when tested with One Step Drug Screen Test Card at a concentration of 100  $\mu$ g/mL.

#### **Non Cross-Reacting Compounds**

Acetaminophen	Maprotiline
Acetophenetidin	MDE
N-Acetylprocainamide	Meperidine
Acetylsalicylic acid	Meprobamate
Aminopyrine	
Amitriptyline	Methoxyphenamine
Amoxicillin	Nalidixic acid
Ampicillin	Naloxone
L-Ascorbic acid	Naltrexone
	Naproxen
Apomorphine	Niacinamide
Aspartame	Nifedipine
Atropine	Norethindrone
Benzilic acid	D-Norpropoxyphene
Benzoic acid	Noscapine
Benzphetamine	DL-Octopamine
Bilirubin	Oxalic acid
(±) – Brompheniramine	
Caffeine	Oxolinic acid
Cannabidiol	Oxymetazoline
Chloralhydrate	Papaverine
Chloramphenicol	Penicillin-G
Chlorothiazide	Pentazocine hydrochloride
(±) – Chlorpheniramine	Perphenazine
Chlorpromazine	Phenelzine
Chlorquine	Trans-2-phenylcyclo-propylamine hydrochloride
	L-Phenylephrine
Cholesterol	$\beta$ -Phenylethylamine
Clomipramine	Phenylpropanolamine
Clonidine	Prednisolone
Cortisone	Prednisone
(-) Cotinine	Promazine
Creatinine	Promethazine
Deoxycorticosterone	DL-Propranolol
Dextromethorphan	D-Propoxyphene
	D-Pseudoephedrine
Diclofenac	Quinacrine
Diflunisal	Quinidine
Digoxin	Quinine
Diphenhydramine	Ranitidine
Doxylamine	Salicylic acid
(-) -? -Ephedrine	Serotonin
$\beta$ -Estradiol	Sulfamethazine
Estrone-3-sulfate	Sulindac
Ethyl-p-aminobenzoate	
[1R,2S] (-) Ephedrine	Tetracycline
(L) – Epinephrine	Tetrahydrocortisone, 3-acetate
Erythromycin	Tetrahydrocortisone 3-( $\beta$ -D-glucuronide)
Fenopufen	Tetrahydrozoline
Furosemide	Thiamine
Gentisic acid	Thioridazine
Hemoglobin	DL-Tyrosine
Hydralazine	
Hydrochlorothiazide	
Hydrocortisone	

O-Hydroxyhippuric acid	Tiobutamide
p-Hydroxyamphetamine	Triamterene
3-Hydroxytyramine	Trifluoperazine
Ibuprofen	Trimethoprim
Imipramine	Trimipramine
Iproniazid	Tryptamine
(±) – Isoproterenol	DL-Tryptophan
Isoxsuprine	Tyramine
Ketamine	Uric acid
Ketoprofen	Verapamil
Labetalol	Zomepirac
Loperamide	

#### **BIBLIOGRAPHY**

1. Stewart DI, T Inoba, M Ducassen, W Kalow. *Clin. Pharmacol. Ther.* 1979; 25:264
2. Ambre J. *J. Anal. Toxicol.* 1985; 9:241
3. Hawks RL, CN Chiang. *Urine Testing for Drugs of Abuse*. National Institute for Drug Abuse (NIDA), Research Monograph 73, 1986
4. Tietz NW. *Textbook of Clinical Chemistry*. W.B. Saunders Company. 1986; 1735
5. FDA Guidance Document: Guidance for Premarket Submission for Kits for Screening Drug: of Abuse to be Used by the Consumer, 1997
6. Robert DeCresce. *Drug Testing in the workplace*, 114
7. Baselt RC. *Disposition of Toxic Drugs and Chemicals in Man*. 2nd Ed. Biomedical Publ. Davis, CA. 1982; 488

#### **BioTechNostiX Inc.**

*www.biotechnostix.com*

*phone: (905) 944-9565*

*fax: (905) 944-0406*

*e-mail: sales@btnx.com*

DN: R015935-0  
Eff. Date: June 4, 200;