

Rapid detection of “synthetic marijuana”
compounds also known as K2 & Spice.

Synthetic Cannabinoids (K2) Urine Test Panel

Single Drug Urine Test Device

Targets 2 federally banned synthetic
cannabinoid compounds in urine:
JWH-018 and JWH-073.

*For in vitro diagnostic use. For use in
employment and insurance testing.*




Synthetic Cannabinoids Urine Test Panel

The Synthetic Cannabinoids (K2) Urine Test Panel offers fast and convenient drug detection intended for employment and insurance testing. The drug test is easy to administer and helps you make informed decisions about abuse of synthetic cannabinoids “K2 & Spice.”

- Screens for the presence of JWH-018 and JWH-073 metabolites in urine
- Simple to use, collect specimen and dip strip to activate test
- Read results in as little as five minutes
- For use in employment and insurance drug testing

Easy to Use

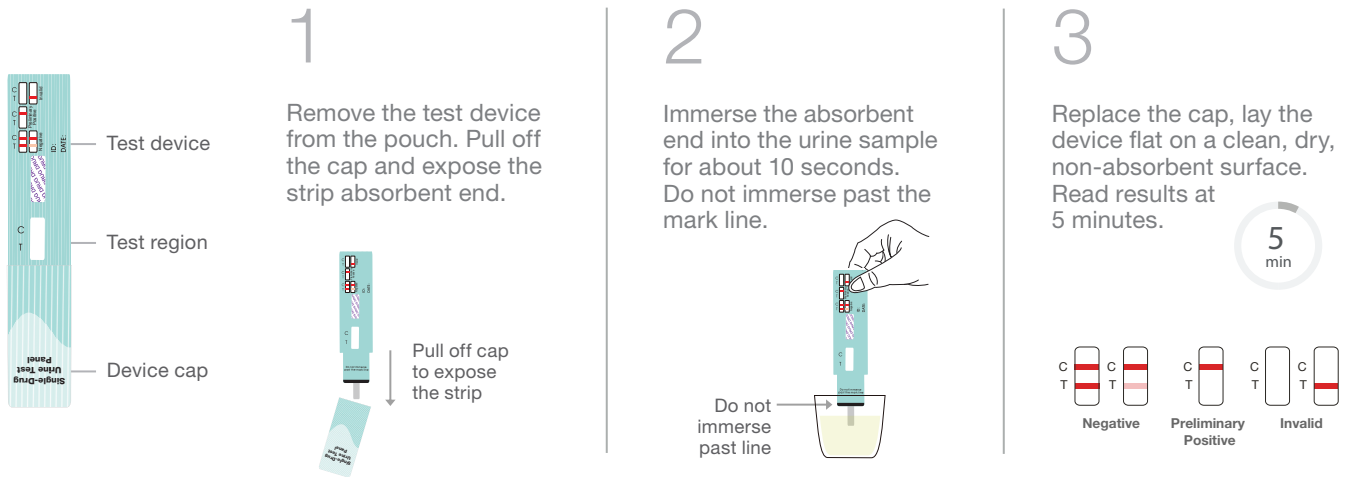

1
Remove the test device from the pouch. Pull off the cap and expose the strip absorbent end.

2
Immerse the absorbent end into the urine sample for about 10 seconds. Do not immerse past the mark line.

3
Replace the cap, lay the device flat on a clean, dry, non-absorbent surface. Read results at 5 minutes.

Labels for the test device: Test device, Test region, Device cap.

Labels for the results: Negative, Preliminary Positive, Invalid.

Timer: 5 min

PRODUCT PROCEDURE NOTE: Refer to product insert for complete instructions, limitations, and warnings. For in vitro diagnostic use. For use in employment and insurance testing. The assay provides a qualitative, preliminary test result. A more specific analytical method must be used in order to obtain a confirmed result. Gas chromatography-mass spectrometry (GC-MS) or liquid chromatography-tandem mass spectrometry (LC-MS/MS) are preferred confirmatory methods. Professional judgment should be applied to any drug test result, particularly when preliminary results are positive.

Ordering Information

Item #	Synthetic Cannabinoids (K2) Urine Test Panel	Cutoff Level	Packaging
ATS-K2	Synthetic Cannabinoids (K2) Targets JWH-018 and JWH-073 metabolites	50 ng/mL	25 tests per box



Alere Toxicology

Distributed By: Blue Grass Drug Screen, Inc. Toll Free 1-866-323-7336