



Dip Card

PROCEDURE CARD

- 1 Remove the test device from the pouch. Pull off the cap and expose the strip absorbent end.**

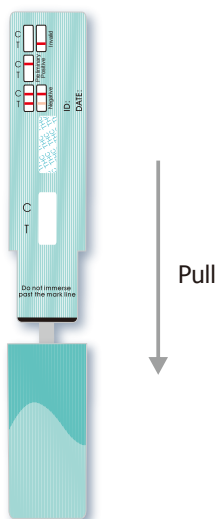

- 2 Immerse the absorbent end into the urine sample for about 10 seconds. Do not allow the urine past the mark line.**

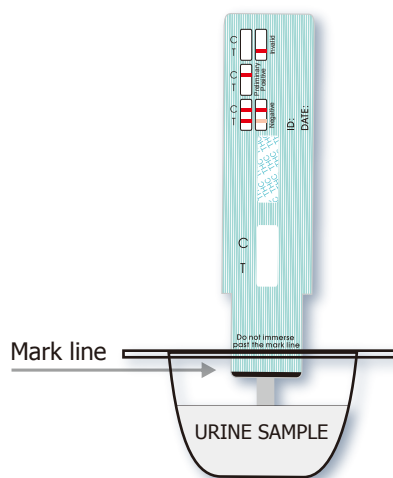


- 3 Replace the cap, lay the device flat on a clean, dry, non-absorbent surface. Read results at 5 minutes.**



Lay the device flat

- 4 Interpretation of Result**

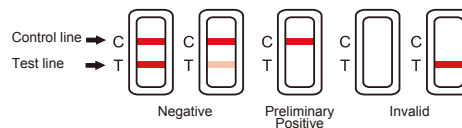

Preliminary Positive

One line appears in the Control region (C) and no line appears in Test region (T).

Negative

Two lines appear. A colored line appears in the Control region (C) and a colored line appears in the Test region (T). The presence of even very faint lines in both regions Control (C) and Test (T) indicates negative results.

Invalid

No lines in both regions Control (C) and Test (T) or a line in the Test (T) region.