



# T-Cup® Compact

# Multi-Drug Urine Test Cup




- **Cost Effective**
- **Reduce storage space**
- **Lower shipping expense**
- **FDA Cleared & CLIA Waived**
- **Same Technical Features with T-Cup®**



## Test Procedure


1

**Donor removes the cup lid.**

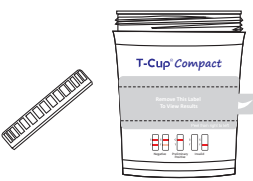


2

**Donor provides urine specimen. Recap the cup and place it on a flat surface.**

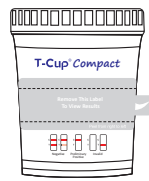


3


**Technician peels off label to view results.**

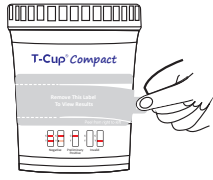

4

**Read adulteration test results between 1-2 minutes.**

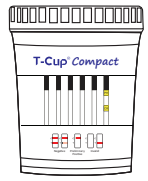




**2 min**


Note: Not all cup configurations come with Adulteration Strip.

Adulteration Strip:  
See color chart and package insert for interpretation.

5


**Read drug strips at 5 minutes. Do not read the test results after 5 minutes.**

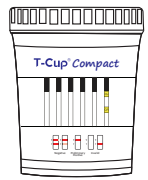





**5 min**

Read results at 5 minutes.

 Negative

 Preliminary Positive

 Invalid


All positive results are presumptive and should be confirmed by an alternative method (e.g. GC/MS)

6







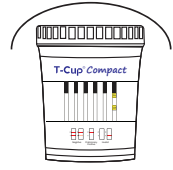

























**Technician dates and initials the security seal and places security seal over the cup lid.**

Date \_\_\_\_\_ Initial \_\_\_\_\_ Security Seal

Place security seal over cup lid



## CLIA Waived Drugs

 AMP	Amphetamine (AMP)	1000ng/mL	 TCA	Tricyclic Antidepressants (TCA)	1000ng/mL
 AMP		500ng/mL	 TCA		
 BAR	Barbiturates (BAR)	300ng/mL	 MET	Methamphetamine (MET/mAMP)	1000ng/mL
 BAR			 MET		500ng/mL
 BZO	Benzodiazepines (BZO)	300ng/mL	 MDMA	Methylenedioxymethamphetamine (MDMA)	500ng/mL
 BZO			 MDMA		
 BUP	Buprenorphine (BUP)	10ng/mL	 MOP	Morphine (MOP/OPI300)	300ng/mL
 BUP			 MOP		
 COC	Cocaine (COC)	300ng/mL	 OPI	Opiate (OPI)	2000ng/mL
 COC		150ng/mL	 OPI		
 THC	Cannabinoids (THC)	50ng/mL	 OXY	Oxycodone (OXY)	100ng/mL
 THC			 OXY		
 EDDP	2-ethylidene-1,5-dimethyl-3,3-diphenylpyrrolidine (EDDP)	300ng/mL	 PCP	Phencyclidine (PCP)	25ng/mL
 EDDP			 PCP		
 MTD	Methadone (MTD)	300ng/mL	 PPX	Propoxyphene (PPX)	300ng/mL
 MTD			 PPX		

Built-in Adulteration Parameters: Creatinine (CR), Specific Gravity (SG) & pH. Oxidant (OX) & Nitrite (NI) available upon request.

Item Code	Description	Complexity
CDOA-254	5-Panel T-Cup Compact; AMP,COC,mAMP,OPI,THC	CLIA WAIVED
CDOA-8104	10-Panel T-Cup Compact; AMP,BUP,BZO,COC,mAMP,MDMA,MTD,OPI, OXY,THC	CLIA WAIVED
CDOA-8105	10-Panel T-Cup Compact; AMP,BUP,BZO,COC,mAMP,MDMA,MOP,MTD,OXY,THC	CLIA WAIVED
CDOA-3124	12-Panel T-Cup Compact; AMP,BAR,BUP,BZO,COC,MDMA,MTD,OPI,OXY,PCP,TCA,THC	CLIA WAIVED
CDOA-6125	12-Panel T-Cup Compact; AMP,BAR,BUP,BZO,COC,mAMP,MDMA,MOP,MTD,OXY,PCP,THC	CLIA WAIVED
CDOA-6125A3	12-Panel T-Cup Compact; AMP,BAR,BUP,BZO,COC,mAMP,MDMA,MOP,MTD,OXY,PCP,THC with 3 AD	CLIA WAIVED
CDOA-9165FTK	16-Panel T-Cup Compact; AMP,BAR,BUP,BZO,COC,mAMP,MDMA,MOP,MTD,OXY,PCP,THC,ETG,FTY,TRA,K2	Forensic Use Only